


Cleaning and Lubrication Procedure for Nosecones

Slowspeed nosecones must be lubricated on a weekly basis.

CLEANING:

1. Clean outside of the nosecone body with alcohol only and wipe dry (point “A”).

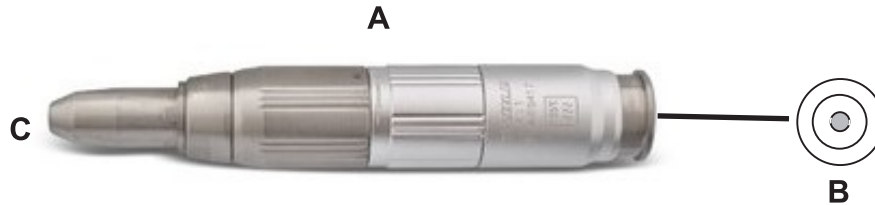

LUBRICATION:

1. Spray lubricant into lower body section for 1 full second. (motor attachment area) (point “B”)
2. Spray lubricant into chuck opening for 1 full second. (point “C”)
3. Insert a mandrel, connect to motor, and run handpiece for 30 seconds.
4. Wipe off excess lubricant using alcohol only.

Handpiece-Turbines.com



high quality dental handpiece turbines for less