

# Cleaning and Lubrication Procedure for Angle Adapters


Slowspeed angle adapters and contra angles must be lubricated regularly.

## CLEANING:

1. Clean outside of angle adapter body and latch head with alcohol only and wipe dry.

## LUBRICATION:

1. Unscrew collar at latch head base to remove head and intermediate shaft from angle adapter lower body section.
2. Spray lubricant into the base of the lower body section for 1 full second. (E-type attachment area) (point “A”)
3. Spray lubricant into the top of the lower body section for 1 full second. (latch head attachment area) (point “B”)
4. Spray lubricant into the latch head base opening and top and bottom of intermediate shaft for 1 full second. (point “C”)
5. Reassemble, connect to motor, and run for 30 seconds.
6. Wipe off excess lubricant using alcohol only.

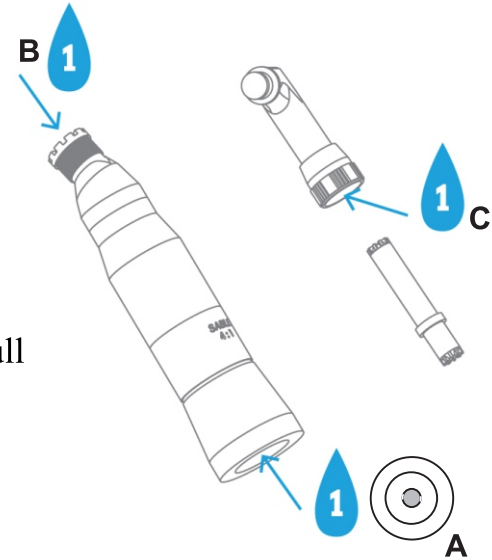

# Handpiece-Turbines.com



high quality dental handpiece turbines for less